

## **BOOMERANG WITHOUT A CHAIN TRACER**

### **VERY IMPORTANT WARNING**

This BOOMERANG Model without a metal tracer is exceptionally made under special request. Normally we take considerable trouble in incorporating a metal tracer. Here is the reason.

Without the chain, if a horn were accidentally superimposed on the anatomy, the artifact thus produced could resemble a tumour or a comparable lesion. This could cause a diagnostic error susceptible of inducing a legal litigation. The use of this chainless filter is therefore done with full knowledge of inherent risks.

We rather suggest the use of a **regular BOOMERANG with a metal chain**, thus preventing unnecessary risks. The chain is normally always projected outside the anatomy, where it is not harmful. Should it be accidentally superimposed, this would be readily recognized. An additional advantage is the verification on the film of the correct position of the BOOMERANG, since the distance between the chain and the skin indicates the amount of filtration applied.

#### **WARNING**

Must be used only by qualified personnel, according to appropriate procedures, and under the responsibility of a physician. OCTOSTOP<sup>®</sup> Inc. and its personnel do not assume any liability regarding the use, indications, consequences, or any situation directly or indirectly related to its products.